

# Clinicare

## Hospital Grade Disinfectant and Decontaminant Cleaner

Kills  
**COVID-19**  
in  
**2**  
minutes



Keep your practice, patients and staff safe with the *newly formulated* Clinicare Hospital Grade Disinfectant and Decontaminant Cleaner



### Improve practice infection control efficiencies

- With only a 2-minute wait time to kill covid-19 change over time between patients is faster. No need to wait 10 minutes like similar products on the market.

### Peace of mind

- Broad Spectrum multi-purpose formula.

### Versatile

- 2 in 1 disinfectant and decontaminant cleaner, replacing the need for different types of wipes and solutions for surface cleaning.

### Safe on Plastics

- Contains food grade plasticiser to maintain plastic, vinyl and upholstery surfaces

### Support local

- Dentalife products are developed and manufactured in Melbourne.

**BACTERICIDAL  
VIRUCIDAL  
FUNGICIDAL  
YEASTICIDAL  
TUBERCULOCIDAL**



### Effective Against

TESTED UNDER DIRTY CONDITIONS

Pseudomonas aeruginosa  
Escherichia coli  
Staphylococcus aureus  
Methicillin Resistant Staphylococcus aureus (MRSA)  
Enterococcus hirae  
Acinetobacter baumannii  
Proteus vulgaris  
Salmonella enterica  
Vancomycin Resistant Enterococcus faecium (VRE)  
Klebsiella pneumoniae  
Poliovirus Type 1  
SARS-CoV-2 (COVID-19)  
Adenovirus  
Herpes Simplex virus Type 1  
Trichophyton mentagrophytes  
Candida albicans  
Candida auris  
Mycobacterium terrae (for TB)

# Clinicare Hospital Grade Disinfectant and Decontaminant Cleaner

Clinicare 2 step process


## STEP 1: Decontaminate



A. Dispense Clinicare towelette



B: Wiping from clean to dirty in an S-shaped pattern, ensuring not to go over the same area twice

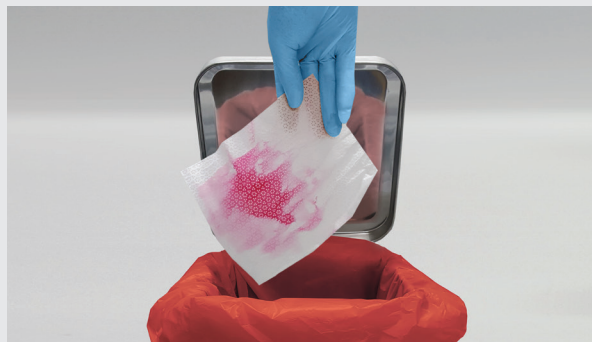

C. Dispose of used towelette

## STEP 2: Disinfect



A. Dispense another Clinicare towelette



B: Wiping the decontaminated area from top to bottom in an S-shaped pattern, ensuring not to go over the same area twice.



C. Dispose of used towelette



Use for rapid decontamination and pre-cleaning of surfaces, instruments and equipment



IPA 70% is the soft alcohol providing antibacterial, cleaning and solvent properties but gentle on skin and surfaces

- DL2901 Clinicare HGD 5L
- DL2902 Clinicare HGD 500ml Squeeze Bottle
- DL2906 Clinicare HGD Trigger 500mL
- DL2910 Clinicare HGD Towelette Canister (220)
- DL2910R Clinicare HGD Towelette Refill (220)
- DL2960 Clinicare HGD ULTRA (180) Towelette Canister



**Manufactured by:**  
 Dentalife Australia Pty Ltd  
 Factory 9 /505 Maroondah Highway  
 Ringwood, Victoria 3134 Australia  
 Phone (03) 9879 1226  
 www.dentalife.com.au