



Clinicare Alcohol Free Hospital Grade Disinfectant New Alcohol Free formula

Composition	Benzethonium Chloride 0.40% w/v, Cetrimide 0.45% w/v, Polyaminopropyl biguanide 0.1% w/v										
Uses	Clinicare Alcohol Free Hospital Grade Disinfectant solution is a highly effective cleaner and disinfectant. It has been developed for use in hospital operating rooms, isolation areas, neonatal units and other critical care areas where environmental control of cross contamination is important.										
Advantages	<ul style="list-style-type: none"> • 1 minute kill time BACTERICIDAL, 30 seconds kill SARS-CoV-2(COVID-19) • Alcohol Free • Broad Spectrum, fast acting • 2 in 1 disinfectant and decontaminant cleaner • Formulated and tested for use in healthcare and critical environments • Safe for use on all surfaces including plastics, metals, ceramics, leather, vinyl, etc. 										
Active Against	<i>Pseudomonas aeruginosa, Escherichia coli, Staphylococcus aureus, Enterococcus hirae, Proteus vulgaris, Salmonella enterica, VRE faecium, Klebsiella pneumoniae, Acinetobacter baumannii, SARS-CoV-2 (COVID-19), Candida albicans</i> TESTED UNDER DIRTY CONDITIONS										
Contraindications	Not to be mixed with any detergents, chemicals or other cleaning agents.										
Precautions	Do not swallow. Avoid contact with skin and eyes. Use undiluted. If visible soiling is evident, surfaces should be pre-cleaned / decontaminated before disinfection.										
Clinical Procedure	<p>For Pre-cleaning / Decontamination of Surfaces Apply Clinicare Alcohol Free Hospital Grade Disinfectant directly to the contaminated surface. Physically wipe surface clean using a Clinicare towelette. Discard any towelette used to decontaminate the area.</p> <p>For Disinfecting Surfaces Apply Clinicare Alcohol Free Hospital Grade Disinfectant directly to the pre-cleaned surface, thoroughly wetting area to be disinfected. Allow surface to remain wet for 1 minute under ambient conditions. Allow surface to air dry or, if required, wipe or rinse clean after 1 minute. When preparing surfaces only use Clinicare towelettes or a lint free cloth. Discard any used towelettes after use.</p>										
Poison Schedule	Not Applicable										
Safety Directions	Avoid contact with eyes. Avoid contact with skin. Wash hands after use.										
First Aid	For advice, contact a Poisons Information Centre (Australia) 13 11 26, (Thailand) +66 (0)2 547 0100										
Regulatory	Australia : Under Consideration Thailand : Manufacturing License No. Sor Phor 15/2558										
Presentations	<table> <tr> <td>80 Dispensing Wipes Canister</td> <td>Product Code : DL2380</td> </tr> <tr> <td>180 Dispensing Wipes Canister</td> <td>Product Code : DL2344</td> </tr> <tr> <td>220 Dispensing Wipes Canister</td> <td>Product Code : DL2345</td> </tr> <tr> <td>750 mL Trigger Spray Pack</td> <td>Product Code : DL2303</td> </tr> <tr> <td>5 L Screw Cap Plastic Container</td> <td>Product Code : DL2301</td> </tr> </table>	80 Dispensing Wipes Canister	Product Code : DL2380	180 Dispensing Wipes Canister	Product Code : DL2344	220 Dispensing Wipes Canister	Product Code : DL2345	750 mL Trigger Spray Pack	Product Code : DL2303	5 L Screw Cap Plastic Container	Product Code : DL2301
80 Dispensing Wipes Canister	Product Code : DL2380										
180 Dispensing Wipes Canister	Product Code : DL2344										
220 Dispensing Wipes Canister	Product Code : DL2345										
750 mL Trigger Spray Pack	Product Code : DL2303										
5 L Screw Cap Plastic Container	Product Code : DL2301										
Storage	Store below 30°C in a cool, dry place.										
Shelf Life	3 years from date of manufacture as indicated on pack.										