



# DART Tablets

## Dentalife Anti-Rust Tablets

<b>Composition</b>	Sodium Nitrite
<b>Uses</b>	Used to prevent and protect dental and medical metallic instruments or containing metallic parts from rusting and corrosion during prolonged exposure to cleaning and disinfectant solutions. Addition to alcohol based disinfectants and storage solutions protects stainless steel and other metal based instruments from corrosion and rusting.
<b>Advantages</b>	<ul style="list-style-type: none"> <li>• Rapid dissolving tablets</li> <li>• Easy dispensing system</li> <li>• High purity Sodium Nitrite</li> </ul>
<b>Contraindications</b>	Not a cleaning agent.
<b>Precautions</b>	See "Safety Directions" below.
<b>Procedure for Use</b>	Add 2 tablets per litre of aqueous solution or 1 tablet per litre of alcoholic /aqueous solution. Ensure tablets are added and fully dissolved prior to addition of instruments.
<b>Poison Schedule</b>	Australia - Schedule 5
<b>Hazardous Goods Codes</b>	UN No. 1500 Class 5.1 Packing Group III
<b>Safety Directions</b>	Oxidizing agent. Contact with combustible materials may cause fire. Toxic if swallowed.
<b>First Aid</b>	If poisoning occurs, contact a doctor or Poisons Information Centre. Phone 13 11 26. If swallowed and more than 15 minutes from a hospital, induce vomiting, preferably using Ipecac Syrup APF.
<b>Presentations</b>	100 x 1gram tablets in secured Plastic Jar      Product Code : DL2401
<b>Storage</b>	Store below 30°C and protect from moisture. Replace lid immediately after use.
<b>Shelf Life</b>	3 years from date of manufacture, indicated on bottle.

Developed and manufactured in Australia by: Dentalife Australia Pty Ltd  
 Factory 9/505 Maroondah Highway, Ringwood, Victoria 3134 Australia  
 Phone : (03) 9879 1226 , Fax : (03) 9879 1376  
 Email : admin@dentalife.com.au, www.dentalife.com.au

