

PRODUCT PROFILE

DentaLife
AUSTRALIA

Krystal Kleen

Active Ingredients	Sulfamic Acid, Citric Acid
Product Description	For use in cleaning of water distillation units. It can also be used to remove mineral deposits on surfaces, including stainless steel.
Product Advantages	Removes mineral deposits to restore surfaces
ARTG Entry	Exempt
Directions for Use	Fill a clean 1 litre container with warm water and add 2 tablespoons of Krystal Kleen. Dissolve and pour into water chamber. Add water to cover any mineral deposits. Allow to stand for 4 – 5 hours or overnight for heavier mineral deposits.
Poison Schedule	Poison
Safety Directions	Avoid contact with eyes and skin.
Warnings	Corrosive
First Aid	If swallowed, do not induce vomiting. If skin or hair contact occurs, remove contaminated clothing And flush skin and hair with running water. If in eyes, hold eyelids apart and flush the eye continuously with running water. Continue to flush until advised to stop by the Poison Information Centre. For advice, contact a Poisons Information Centre 13 11 26 for Australia 0800 764 766 for New Zealand or call a doctor at once.
Storage	Store in a dry place below 30 °C
Shelf life	3 years from date of manufacture as indicated on pack.
Product Configuration	DL7100 Krystal Kleen 250 g jar



Developed and Manufactured in Australia by: Dentalife Australia Pty Ltd
9/505 Maroondah Hwy Ringwood, Vic 3134
www.dentalife.com.au