

PRODUCT PROFILE

DentaLife
AUSTRALIA

ActivEtch 37% Gel & Solution

Active Ingredients	Phosphoric Acid 37% w/v
Product Description	ActivEtch is a 37% phosphoric etchant solution used for etching enamel and dentine prior to applying bonding adhesives and placing composite restorations or dental sealants. ActivEtch is self-limiting to ensure over etching does not occur. ActivEtch is designed for quick and complete removal upon rinsing.
Product Advantages	<ul style="list-style-type: none">• Rapid action which terminates after effective etching• Contrasting soluble, blue dye• Rinses off easily with water• Self-limiting etchant gel
ARTG Entry	160312
Directions for Use	<ol style="list-style-type: none">1. Remove cap from syringe and securely attached a blue dispensing tip.2. Confirm the flow of etchant by dispensing a small amount onto a mixing pad.3. Isolate the tooth or teeth.4. Apply etchant to enamel and dentine for 15 seconds.5. Rinse thoroughly, dry and proceed to bond the tooth as per the manufacturer's instructions.6. Re-cap, disinfect and wipe syringe with a Clinicare Hospital Grade Disinfectant between uses. If syringe is cover by a barrier sleeve: remove the tip, re-cap, and discard the barrier sleeve.7. Dispose of tip.
Precautions	If product comes into contact with soft tissue, rinse area with copious amounts of water. Near pulp exposures should be treated with a protective base prior to placing ActivEtch.
Warnings	To avoid cross contamination, do not re-use tips.
First Aid	For advice, contact the Poison Information Centre 13 11 26 for Australia or a doctor (at once). If swallowed, do not induce vomiting. If in eyes, hold eyelid apart and flush the eye continuously with running water. Continue flushing until advised to stop by a Poison information Centre. If skin or hair contact occurs, remove contaminated clothing and flush skin and hair with running water.
Storage	Store above 4 °C and below 30 °C , do not freeze.
Shelf life	3 years from date of manufacture as indicated on pack.
Product Configuration	DL2503 ActivEtch 10 x 3g Syringe Kit DL2511 ActivEtch Solution 30 mL bottle DL2560 ActivEtch 60 mL Bulk Syringe DL2566 ActivEtch 60 mL bulk Syringe Starter Kit



Developed and Manufactured in Australia by: Dentalife Australia Pty Ltd
9/505 Maroondah Hwy Ringwood, Vic 3134
www.dentalife.com.au