

PRODUCT PROFILE

DentaLife
AUSTRALIA

Silane Bond Enhancer

Description	Silane is a single-component solution used to promote increased bond strength between organic resins and porcelain or ceramic surfaces.
Indications	Silane is necessary for bonding support after porcelain etching.
Contraindications	Intraoral use in difficult or uncooperative patients.
Clinical Procedure	<ol style="list-style-type: none">1. Carefully follow instructions for Porcelain Etch and treat porcelain or ceramic surfaces.2. Neutralize, rinse and dry. Ensure surface to be treated is now completely dry. Residual moisture will adversely affect bonding.3. Connect a 23G Dispenser tip to the Silane syringe and test dispensing pressure by expressing a small amount onto a mixing pad to familiarize the flow characteristic of the syringe.4. Next, apply Silane with a mini brush tip and leave to evaporate for 60 seconds. Air dry with oil-free air if not completely dry.5. Remove tip and re-cap syringe immediately after use. Dispose of tip.6. Apply your preferred restorative composite or bonding resin according to product (manufacturer's) instructions.
Precautions	<ol style="list-style-type: none">1. Avoid contact with skin. Do not swallow.2. Operator, Assistant and Patient must wear protective eyewear.3. Reseal (recap) syringe immediately after use. Store capped at all times.4. Silane may set in tip after a few hours. Remove tip immediately after use and recap syringe.
First Aid	For advice, contact a Poisons Information Centre (Australia) 13 11 26, (Thailand) +66 (0)2 547 0100
Poison Schedule	Not Applicable
Important Notes	<ol style="list-style-type: none">1. Review clinical procedure (above), precautions and MSDS before beginning application.2. For Professional dental use only.3. Keep products away from direct sunlight and heat sources.4. Test dispensing of materials from the syringe and tip before intraoral application.5. Never force syringe plungers.
Presentations	2 x 2.5 mL Syringe Kit with 10 x 23G Dispenser Tips Product Code : DL1711
Storage	Store below 25°C in a cool, dry place.
Shelf Life	2 years from date of manufacture as indicated on pack.

Developed in Australia by : Dentalife Australia Pty Ltd

www.dentalife.com.au

Manufactured by : Pharmadesign Co., Ltd.

199/3 Moo 21, Bangpleeyai Bangplee Samutprakarn Thailand 10540

PD Version 1 Issued: 04/2021