

Optum F Prophypaste

Composition	Medium / Regular grit Alumina based prophypaste Freshmint, Wildberry and Tropical flavours Includes Sodium Fluoride 2.2mg/g (0.22% w/w)
Uses	Optum F Prophypaste is for removal of surface stains, pellicle and calculus on teeth not removed through normal patient brushing and oral hygiene. Specifically formulated for mild abrasion (polishing) and cleaning of tooth structures. Optum F is also a paediatric prophypaste which provides a supplementary treatment of topical fluoride to tooth surfaces. This assists in prevention of caries and sensitivity on the newly exposed surfaces.
Advantages	<ul style="list-style-type: none"> • Contains 0.22% Sodium Fluoride for topical application • High density, non-splatter formulation • Triple Action - • 1/ Effects abrasion, 2/ Dissolving of surface stains and 3/ Topical F application • Exciting fresh flavour alternatives • Doesn't dry-out, maintains consistency • Presented in convenient, low-form, plastic jar • Available in single dose prophy cups
Contraindications	Do not use to clean teeth prior to restorative procedures or other procedures requiring adhesion to tooth surfaces.
Precautions	Eye protection should be worn when using the paste.
Clinical Procedure	Take a small amount of Optum F Prophypaste and place onto a rubber cup or bristle brush attached to a slow speed dental handpiece. Work the paste from the cervical margins over the body of the tooth until staining or build up is removed. Reapply as required. Use dental floss to remove any grit remaining between teeth. If using Optum F prophy paste single dose cups: Place an individual prophy paste cup into the prophy ring and remove the foil seal. Using a rubber cup or bristle brush take a small amount of prophy paste by inserting into the dose cup and whilst running the handpiece at low speed. Work paste over the teeth.
Poison Schedule	Not Applicable
First Aid	If poisoning occurs contact a doctor or Poisons Information Centre. Phone 13 11 26. If in eyes, hold eyes open, flood with water for at least 15 minutes and see a doctor.
Presentations	DL0211 Optum F prophy paste 200 gm jar Wildberry DL0212 Optum F prophy paste 200 gm jar Freshmint DL0214 Optum F prophy paste 200 gm jar Tropical DL0243 Optum F prophy cups 2g + 5 rings Freshmint DL0244 Optum F prophy paste cups 2 g + 5 rings Wildberry
Storage	Store below 30 °C in a cool, dry place.
Shelf Life	2 years from date of manufacture, as indicated on pack.

