



SPOT WHAT YOU'VE MISSED

WORLD FIRST PLAQUE DISCLOSING TOOTHPASTE



FOR THE
CLEANEST
TEETH EVER

PlaqPro™

EVERYDAY USE

NEW PLAQUE DISCLOSING TOOTHPASTE SYSTEM

The PlaqPro toothpaste and torch system helps you maintain the cleanest teeth and healthiest gums possible. PlaqPro toothpaste contains Fluorescein, a safe and effective plaque stain, which will only be visible under the blue light supplied. This will show you where further brushing should aim, if required, to remove ALL plaque. PlaqPro was developed in conjunction with Australian Dentists to provide truly effective cleaning of your teeth every time you brush.

DIRECTIONS FOR USING PLAQPRO

To get complete removal of plaque from your teeth and tongue.

BRUSH TEETH normally using PlaqPro toothpaste.

RINSE THOROUGHLY with water (it is recommended not to swallow as with all toothpaste products).

SCAN YOU TEETH using the blue light torch while looking in a mirror. Remaining plaque will illuminate when exposed to the light.

REMOVE any remaining plaque by brushing or flossing the identified places.

TOOTHBRUSH - use the blue light to check your toothbrush (in the bristles). Run under hot water to remove plaque.

PLAQUE FREE NOW - Enjoy fresher breath, cleaner teeth, healthier gums and a clean toothbrush next time you brush using PlaqPro.

PLAQPRO ENQUIRIES plaqpro@dentalife.com.au

MADE IN AUSTRALIA BY Dentalife Australia Pty Ltd. 9/505 Maroondah Hwy, Ringwood, VIC, 3134. www.dentalife.com.au PlaqPro is a trademark of Dentalife Australia Pty Ltd.

INGREDIENTS Sorbitol, Aqua, Hydrated Silica, Xylitol, Sodium Lauryl Sulphate, Xanthan Gum, Titanium Dioxide, Cellulose Gum, Flavour, Fluorescein Sodium, Saccharin, Sodium, Sodium Fluoride, C173360. Preservative Free.

PlaqProTM
EVERYDAY USE

