

## PRODUCT PROFILE

**DentaLife**  
AUSTRALIA

# ENDO VIT

<b>Composition</b>	Hydrocarbon propellant (Cryo Spray -55°C for Pulp Vitality Testing)
<b>Uses</b>	As an aid to testing pulpal vitality rapid cooling of impression materials, eg. Hydrocolloids, etc. For professional use only.
<b>Advantages</b>	<ul style="list-style-type: none"><li>• Freezes cotton pledget, bud or roll down to -55°C</li><li>• Easy and safe to use</li><li>• Cost effective system</li></ul>
<b>Contraindications</b>	Unwilling patient or hypersensitivity to cold or heat.
<b>Precautions</b>	Do not spray directly onto tooth or skin.
<b>Clinical Procedure</b>	Use the extension tube provided and insert into the nozzle of the actuator. Direct the extension tube at a small cotton pledget, cotton bud or cotton roll held at the end of the tube and spray the propellant onto the target for 3 to 4 seconds. Allow the treated cotton surface to freeze (10 to 20 seconds). Once frozen, place carefully onto the surface of the suspect tooth and assess the response. Repeat as necessary.
<b>Poison Schedule</b>	Not applicable.
<b>First Aid</b>	For advice, contact a Poisons Information Centre (Australia) 13 11 26, (Thailand) +66 (0)2 547 0100
<b>Regulatory</b>	Australia : ARTG 186775
<b>Presentation</b>	Aerosol Can with Dispensing Tube attached 200 mL Product Code : DL6400
<b>Storage</b>	Protect from direct sunlight and do not expose to temperatures exceeding 50°C. Keep in a cool place away from heat. Do not pierce or burn can, even after use.
<b>Shelf Life</b>	3 years from date of manufacture as indicated on pack.

Developed in Australia by: **Dentalife Australia Pty Ltd**

[www.dentalife.com.au](http://www.dentalife.com.au)

Distributed by: **Pharmadesign Co., Ltd.**

199/3 Moo 21, Bangpleeyai Bangplee Samutprakarn Thailand 10540

PD Version 1 Issued: 10/2020