

## PRODUCT PROFILE

**DentaLife**  
AUSTRALIA

# Chlorhexidine 0.2%

<b>Active Ingredients</b>	Chlorhexidine Gluconate 2mg/mL
<b>Product Description</b>	For use in endodontic file cleaning protocols
<b>Product Advantages</b>	Chlorhexidine solution, no additives Pure Chlorhexidine solution prepared to ensure safety and efficacy for intended purpose. Formulated specifically as per the cleaning protocol published: P. Parashos, Linsuwanont, Messer, A cleaning protocol for rotary nickel-titanium endodontic instruments. Australasian Dental Journal 2004;49: (1)20-27
<b>ARTG Entry</b>	Exempt
<b>Directions for Use</b>	Soak a scouring sponge in Chlorhexidine 2% solution. Once sponge has been soaked, clean the endodontic instrument with vigorous strokes to ensure all debris is removed from the instrument.
<b>Poison Schedule</b>	Exempt
<b>Safety Directions</b>	Avoid contact with eyes and skin.
<b>Warnings</b>	-
<b>First Aid</b>	For advice, contact a Poisons Information Centre (e.g. phone Australia 13 11 26; New Zealand 0800 764 766) or a doctor (at once).
<b>Storage</b>	Store Below 25 °C and protected from light
<b>Shelf life</b>	2 years as stated on the pack
<b>Product Configuration</b>	DL5500 Chlorhexidine 0.2% 500 mL



Developed and Manufactured in Australia by: Dentalife Australia Pty Ltd  
9/505 Maroondah Hwy Ringwood, Vic 3134  
[www.dentalife.com.au](http://www.dentalife.com.au)