



Clinicare

Hospital Grade Disinfectant

Composition	0.28% Benzethonium Chloride, 18% Isopropyl Alcohol
Wipe Composition	Clinicare Hospital Grade Disinfectant wipes are available in 220 Wipe Canister and Refill - 25 gsm Spunbond (15 x 23 cm) 180 Wipe Canister and Refill - 38 gsm Meltblown (15 x 17 cm) 200 Wipe Flat Pack - 30 gsm Meltblown (18.5 x 26 cm)
Uses	Clinicare Hospital Grade Disinfectant is used for rapid decontamination and disinfection of hard, non-porous surfaces. Broad spectrum, multi-purpose cleaner/disinfectant for use on plastics, stainless steel, vinyl, upholstery, painted surfaces, glass, etc. Supports infection control procedures in hospitals, surgeries, clinics and other active environments.
Advantages	<ul style="list-style-type: none"> • 1 minute kill time BACTERICIDAL, VIRUCIDAL, FUNGICIDAL, YEASTICIDAL, TUBERCULOCIDAL • Broad Spectrum activity • Formulated in Australia for the use in healthcare and critical environments • 2 in 1 cleaner and disinfectant • Contains food grade plasticiser to maintain plastic, vinyl and upholstery surfaces • Streak free, smear free, low foaming, non-corrosive solution
Active Against	<i>Pseudomonas aeruginosa, Escherichia coli, Staphylococcus aureus, Methicillin Resistant Staphylococcus aureus (MRSA), Enterococcus hirae, Acinetobacter baumannii, Proteus vulgaris, Salmonella enterica, Vancomycin Resistant Enterococcus faecium (VRE), Klebsiella pneumoniae, Poliovirus Type 1, SARS-CoV-2 (COVID-19), Adenovirus, Herpes Simplex virus Type 1, Influenza A virus (H3N2), Influenza A (H1N1), Influenza B, Trichophyton mentagrophytes, Candida albicans, Candida auris and Mycobacterium terrae (for TB)</i> TESTED UNDER DIRTY CONDITIONS
Contraindications	Not to be mixed with any detergents, chemicals or other cleaning agents.
Precautions	Do not swallow. Avoid contact with skin and eyes. Use undiluted. If visible soiling is evident, surfaces should be pre-cleaned/decontaminated before disinfection.
Clinical Procedure	For Pre-cleaning/Decontamination of Surfaces Decontaminate the surface using Clinicare Alcohol Free, Neutral Detergent or Hospital Grade Disinfectant to reduce contamination and prepare the surface. Wipe the surface from clean to dirty in an S-Shaped pattern, ensuring not to go over the same area twice. Dispose of towelette. To Disinfect Surfaces Apply Clinicare Hospital Grade Disinfectant solution directly to the pre-cleaned surface, thoroughly wetting area to be disinfected. Allow surface to remain wet for 1 minute under ambient conditions. Allow surface to air dry or, if required, wipe or rinse clean after 1 minute. Discard any used towelettes after use.
Poison Schedule	Not Applicable
Safety Directions	Avoid contact with eyes. Avoid contact with skin. Wash hands after use.
First Aid	For advice, contact a Poisons Information Centre (Australia) 13 11 26
Regulatory	Australia : ARTG 335416
Presentations	DL2901 Clinicare HGD 5 L DL2906 Clinicare HGD 500 mL DL2960 Clinicare HGD Towelette Canister (180 wipes) DL2960R Clinicare HGD Towelette Refill (180 wipes) DL2970 Clinicare HGD Flat Pack (200 wipes)
Storage	Store below 25°C in a cool, dry place.
Shelf Life	3 years from date of manufacture as indicated on pack.

