

DentaLife

Achieve hemostatis faster with 20% Ferric Sulfate



FerriStat hemostatic gel is recommended for use during crown and bridge procedures. It is great for restorations where bleeding needs to be controlled.

- 20% Ferric Sulfate gel
- Stops the bleeding immediately, saving procedure time.
- Eliminates sulcular fluid contamination, ensuring the best bonding outcomes.
- Leaves no residue and no interference with impression material.
- Gel like consistency, will not run or drip
- Convenient syringe delivery system.



Reorder Codes:


FerriStat – 20% Ferric Sulfate Gel
DL2580 FerriStat 10 x 1.2 mL Syringes & 20 infusor tips
DL2585 FerriStat: 30 mL Syringe & 10 Infusor tips
Infusor Tips
DL2588 Infusor Tips x 20
DL2589 Infusor Tips x 100



Dentalife Australia Pty Ltd
Factory 9/505 Maroondah Highway Ringwood,
Victoria 3134 Australia
Phone (03) 9879 1226 www.dentalife.com.au

DentaLife

AlStat the perfect hemostatic gel for the aesthetic zone

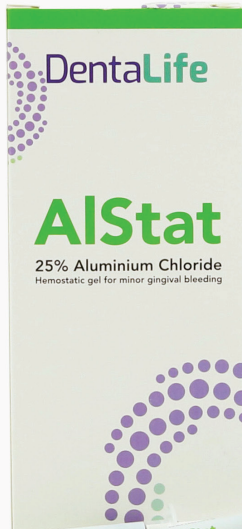


AlStat is a hemostatic gel which effectively eliminates minor gingival bleeding. AlStat hemostatic gel is recommended for anterior restorations and for use during crown and veneer preparations where bleeding needs to be arrested.

- 25% Aluminium Chloride gel
- Eliminates minor bleeding.
- Does not leave any residue that may interfere with bonding or impression materials.
- No staining thanks to a clear solution
- Gel like consistency, will not run or drip.
- Convenient syringe delivery system




Technique

Step 1

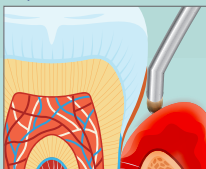


Scrub FerriStat or AlStat gel firmly against the sulcus until bleeding is controlled.

Step 2

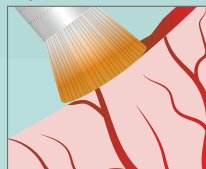


Use an Infusor tip to apply gentle pressure, allowing the hemostatic gel to penetrate the cut capillaries.

Step 3

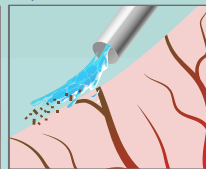

Use air/water to remove coagulum. If bleeding persists, repeat application.

Step 4

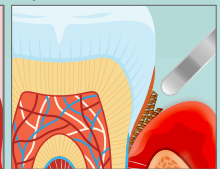

Once hemostasis has been achieved, pack retraction cord as needed.

Reorder Codes:

AlStat - 25% Aluminium Chloride
DL2590 AlStat10 x 1.2 mL Syringes & 20 Infusor tips
DL2595 AlStat 30 mL Syringe & 10 tips

Infusor Tips

DL2588 Infusor Tips x 20
DL2589 Infusor Tips x 100



Dentalife Australia Pty Ltd
Factory 9/505 Maroondah Highway Ringwood,
Victoria 3134 Australia
Phone (03) 9879 1226 www.dentalife.com.au