## Dentalife



## Clinicare IPA 70%

| Active Ingredients | Isopropyl Alcohol 70% v/v  |
|--------------------|--|
| Wipe Composition   | Clinicare IPA 70% is available in Spunbond material 220 Towelette in Canister and Refill 25 gsm (15 x 23 cm)   |
| Uses               | Dentalife IPA 70% is a "soft" alcohol providing antibacterial, cleaning and solvent properties for disinfection of surfaces. It is gentle on skin. Disinfects without leaving residues.  |
| Advantages         | Broad spectrum anti-bacterial activity     Rapid antibacterial activity     Evaporates after use without leaving residues     Available in clear and blue liquid.  |
| Active Against     | Pseudomonas aeruginosa (1 min), Staphylococcus aureus (1 min), Enterococcus hirae (1 min),<br>Candida albican (1 min)  |
| Contraindications  | Not to be added to other cleaning or disinfectant solutions unless indicated by manufacturer.  |
| Precautions        | Avoid contamination of food. Avoid direct contact with eyes as may cause eye irritation. In case of direct eye contact, immediately flush eyes with plenty of water for at least 15 minutes. If irritation persists, seek medical attention  |
| Clinical Procedure | Surfaces to be disinfected must be pre-cleaned. For Surfaces Spray or wipe Clinicare IPA 70% onto surfaces to be disinfected. Thoroughly wet surfaces. Leave wet for up to 1 minutes. Wipe liquid off the surface using a clean, single use towelette or cloth. For Instruments and Equipment Spray or wipe Clinicare IPA 70% onto the instrument or equipment to be disinfected. Follow manufacturer's instructions for disinfecting the instrument or equipment. Leave solution on instrument or equipment for up to 1 minutes. The item can be left to air dry or wiped dry with a single use towelette or lint free cloth. |
| Poison Schedule    | Not Applicable   |
| Warning            | Flammable  |
| Safety Directions  | Avoid contact with eyes. Avoid contact with skin. Wash hands after use.  |
| First Aid          | For advice, contact a Poisons Information Centre (Australia) 13 11 26  |
| Regulatory         | Australia : ARTG No. 391996  |
| Presentations      | DL0920 Clinicare IPA 70% Towelette Canister (220 wipes) DL0920R Clinicare IPA 70% Towelette Refill (220 wipes) DL0901 Clinicare IPA 70% Clear Soultion 5 L DL0911 Clinicare IPA 70% Blue Solution 5 L  |
| Storage            | Store below 25 <sup>o</sup> C in a cool, dry place.  |
| Shelf Life         | IPA 70% Clear & Blue Solution 3 years from date of manufacture IPA 70% Towelette 2 years from date of manufacture  |