



Clinicare Instrument Cleaner

Composition	Quaternary Ammonium Compounds and Corrosion Inhibitors
Uses	Clinicare Instrument and surgical instruments prior to further reprocessing. Prevents soiling during procedures from drying or fixing prior to reprocessing. Assists cleaning and provides contamination control for instruments including extracting forceps, root elevators, dental mirrors, rigid and flexible endoscopes, dental rotating instruments, burs, etc.
Advantages	<ul style="list-style-type: none"> • Suitable for all instruments • Reduces cleaning time and improves staff efficiency • Prevents bacterial growth on contaminated instruments prior to reprocessing • Contains corrosion inhibitors to protect instruments from rusting or pitting • Effective cleaner/disinfectant that can be used in soaking tanks and ultrasonic cleaners • Bactericide, conforms to EN13727 • Economical concentrate format
Contraindications	Not to be mixed with any detergents, chemicals or other cleaning agents.
Precautions	Keep container tightly closed. Keep only in original container. Wash hands thoroughly after handling.
Clinical Procedure	<ol style="list-style-type: none"> 1. Prepare 1 Litre of 2% solution. Dispense 20 mL of Instrument Cleaner concentrate into a 1L jug and make to volume with water. 2. Immerse the contaminated instruments in the solution and leave soaking for a minimum of 15 minutes. 3. Carefully remove the contaminated instruments from the solution and inspect. Used a soft brush, if necessary, to remove any remaining visible debris. 4. Rinse instruments under running water or by immersion in clean water. 5. Dry instruments with a clean disposable paper towel. 6. Pack the instruments and prepare for disinfection or sterilization.
Poison Schedule	Schedule 5 according to SUSMP (Australia)
Safety Directions	Avoid contact with eyes and skin. Wash hands thoroughly after handling.
First Aid	If on skin, wash with plenty of soap and water. If in eyes, wash out immediately with water. Remove contact lenses, if present and easy to remove. Continue rinsing. If eye irritation persists: Get medical advice/attention. For advice, contact the Poisons Information Centre (Australia) 13 11 26, (Thailand) +66 (0)2 547 0100.
Regulatory	Australia : ARTG No. 330307
Presentations	DL2801 Clinicare Instrument Cleaner 5 L
Storage	Store below 30°C in a cool, dry place.
Shelf Life	3 years from date of manufacture as indicated on pack.

