



Clinicare Suction Cleaner DW

Composition	Quaternary ammonium compounds, Corrosion inhibitors and Antifoaming agents
Uses	Liquid concentrated for simple preparation of solution for daily and weekly flushing of <ul style="list-style-type: none"> • Aspiration systems • Amalgam separators • Spittoons
Advantages	<ul style="list-style-type: none"> • Three actions – sanitises, decontaminates and deodorises • No disruptive foam • Alkaline cleaner for removal of daily and weekly deposits • Economical concentrate for preparation of : <ul style="list-style-type: none"> ~ 2% dilution for daily cleaning and deodorising ~ 6% dilution for weekly maintenance and decontaminates • Aldehyde free, Phenol free, Chlorine free • Safe and simple to use
Contraindications	Not to be mixed with any disinfectant or other cleaning agents.
Precautions	Wear protective gloves and gowns and protect the eyes and face. Wash hands after handling and remove gloves. Keep the product in tightly closed container in a cool, dry place.
Clinical Procedure	<p>Prior to treatment: Rinse the suction unit thoroughly with water after surgical procedures and prior to disinfection and cleaning.</p> <p>Daily - Prepare 2% solution.</p> <ol style="list-style-type: none"> 1. Dispense 20 mL of Clinicare Suction Cleaner DW concentrate into 1 L of warm water, 50-60 °C preferred. 2. Aspirate the prepared solution when necessary. Preferable in the middle and end of day. 3. Aspirate via the high and low volume suction tubes. 4. After aspiration, leave the treated system for at least 1 hour or preferably overnight. 5. Rinse thoroughly with 2 litres of clean water prior to reuse. <p>Weekly – Prepare 6% solution</p> <ol style="list-style-type: none"> 1. Dispense 60 mL of Clinicare Suction Cleaner DW concentrate into 1 L of warm water, 50-60 °C preferred. 2. Aspirate the prepared solution at the end of the working week. 3. Aspirate via the high and low volume suction tubes. 4. After aspiration, leave the treated system until resuming practice. 5. Once ready to resume, rinse thoroughly with 2 Litres of clean water <p>The spittoon should also be sanitised with 250 mL of pre-made solution at the same time as the suction and aspiration system.</p> <p>To dispense the solution from the chamber bottle:</p> <ol style="list-style-type: none"> 1. Place the bottle on a steady surface and slightly unscrew the black cap. Do not remove the cap completely. 2. Squeeze the bottle gently to force the solution into the measuring chamber at the top of the bottle, until it reaches the desired amount, marked in mls. 3. If too much solution has been dispensed, pour back into the bottle by tipping the bottle towards the small tube on the side of bottle. 4. Unscrew the black cap and dispense the solution into 1 L of warm water.
Poison Schedule	Not Applicable
Warning	Strongly alkaline
Safety Directions	Avoid contact with eyes. Avoid contact with skin. Wash hands after use.
First Aid	If on skin, wash with plenty of soap and water. If in eyes, rinse continuously with water for several minutes. Remove contact lenses, if present and easy to remove. Continue rinsing. If eye irritation persists: Get medical advice/attention. For advice, contact a Poisons Information Centre (Australia) 13 11 26.
Regulatory	Australia : ARTG No. 330305
Presentations	DL2711 Clinicare Suction Cleaner DW 1L Chamber Bottle
Storage	Store below 30 °C in a cool, dry place.
Shelf Life	3 years from date of manufacture as indicated on pack.

